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1K52
INSTRUCTIONS FOR USE
Deliver copy to the patient

1- DESCRIPTION AND FUNCTIONS

Attached to above knee part of a prosthesis, the **1K52 Rotator** enables the below knee part of a prosthesis to rotate up to 360° whenever the user needs, for example when putting a shoe or sitting on the floor.

The rotator works as follows :

- Press the push button (Fig.1) of the **1K52 Rotator** to release the lock in rotation.
- When rotating back to locked position, rotate the below-knee part gently holding the knee or the foot with your hand. It will be automatically locked again when the knee rotational angle comes back to the original position.

 The rotator should be in the locked position before ambulation to prevent any involuntary motion.

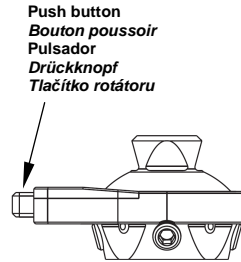
Fig. 1

2- INDICATIONS, ACTIVITY LEVEL, MAXIMUM WEIGHT

The **1K52 Rotator** is designed and tested for everyday activities. It is appropriate for patients with low to high activity level (levels 1 to 4) but not for sports.

This component is appropriate for users whose weight, carried load included, does not exceed 125 kg. This product has been tested according to standard NF EN ISO 10328-2006 for load level P6.

 If used to a person who weighs over 125 kg, damage of parts may occur with the risk for the patient to fall down.

3- HOW TO ASSEMBLE

After making an appropriate alignment of the prosthesis, put appropriate amount of anti-loosening adhesive (XC047) on the four M8 hexagonal screws attached to the product, and tighten them at the torque engraved on the product (15 Nm).

 Place the 1K52 Rotator push button outward facing.

4- MAINTENANCE AND CLEANING

Do not use any solvents on the **1K52 Rotator**.

The use temperature range is -20°C to +50°C.

No maintenance operation such as lubrication, work on the screws or other parts is required, except for the operations indicated in « 3- HOW TO ASSEMBLE ».

 Avoid exposing the rotator to situations that may cause corrosion of the metallic parts (fresh water, sea water, acids, ...).
In case of dysfunction, for example abnormal noise, screw loosening, control and stability problem, or if the rotator sustained a hard shock, please contact your CPO.

5- RECYCLING

This product includes items made of aluminum and steel. They must be recycled in compliance with the laws in force.

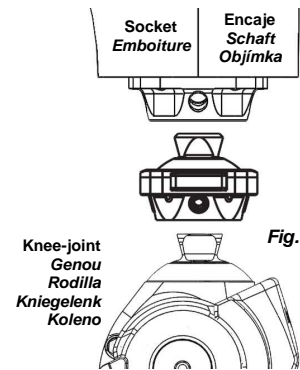
Fig. 2

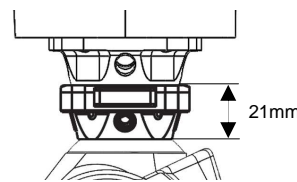
Fig. 3

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